# Crossover Nio

## User Manual

Be well, Live well  
**Wellell**

# CONTENTS

|                                                         |     |
|---------------------------------------------------------|-----|
| <b>ALTERNATING PRESSURE REDISTRIBUTION SYSTEM</b>       |     |
| USER MANUAL                                             | 1   |
| <b>SISTEMA DE REDISTRIBUCIÓN DE PRESIÓN ALTERNA</b>     |     |
| MANUAL DE INSTRUCCIONES                                 | 25  |
| <b>SISTEMA DE REDISTRIBUIÇÃO DE PRESSÃO ALTERNADA</b>   |     |
| INSTRUÇÕES DE UTILIZAÇÃO                                | 50  |
| <b>SYSTÈME DE REDISTRIBUTION À PRESSION ALTERNÉE</b>    |     |
| INSTRUCTIONS D'UTILISATION                              | 74  |
| <b>ALTERNIERENDES DRUCKUMVERTEILUNGSSYSTEM</b>          |     |
| GEBRAUCHSANLEITUNG                                      | 98  |
| <b>ALTERNEREND DRUKSYSTEEM</b>                          |     |
| GEbruikersHANDLEIDING                                   | 123 |
| <b>SISTEMA DI RIDISTRIBUZIONE A PRESSIONE ALTERNATA</b> |     |
| MANUALE DI ISTRUZIONI                                   | 148 |

**MODEL NO.: 9P-077470**

**PLEASE READ THE MANUAL BEFORE USE**

# IMPORTANT SAFEGUARDS

## READ ALL INSTRUCTIONS BEFORE USING

### **DANGER - To reduce the risk of electrocution:**

- (1). Always unplug this product immediately after using it.
- (2). Do not use while bathing.
- (3). Do not place in or drop into water or other liquids.
- (4). Do not place or store this product where it can fall or be pulled into a tub or sink.
- (5). Do not reach for a product that has fallen into the water. Unplug immediately.

### **WARNING - To reduce the risk of burns, electrocution, fire, or injury to persons:**

- (1). Any serious incident that has occurred to a user or patient in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user or patient is established.
- (2). Never perform servicing or maintenance to the device while it is in use.
- (3). Never disassemble this device on your own. Repairs and internal servicing should only be performed by an authorized technician.
- (4). Evaluate patients for entrapment risk according to protocol and monitor patients appropriately.
- (5). The product may be used with caution for patients with spinal injuries, but it is recommended that you consult with a physician before use. However, it should not be used for patients with unstable spinal fractures.
- (6). Close supervision is necessary when this product is used on or near children. Electrical burns or choking accidents may result from a child swallowing small parts if they are detached from the device.
- (7). Use this product only for its intended use as described in this manual. Do not use another mattress not recommended by the manufacturer.
- (8). Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to your supplier or Wellell Inc. for examination and repair.
- (9). Keep the cord away from heated surfaces.
- (10). Never block any air openings of this product or place it on soft surfaces, such as a bed or couch, where openings may be blocked. Keep the air opening free of lint, hair, and other similar particles.
- (11). Never drop or insert any object into any opening or hose.
- (12). No modification of this equipment is allowed unless it is conducted by the authorized technician.
- (13). Mattress covers have passed skin sensitization and skin irritation tests. If you suspect that you may have had or are having an allergic reaction, please consult a physician immediately.
- (14). Do not leave long lengths of tubing around the top of your bed. It could lead to strangulation.
- (15). Keep the pump away from flammable liquids or gases.
- (16). Do not use the extension cord for an extended period of time.
- (17). If there is a possibility of electromagnetic interference with mobile phones, please increase the distance (3.3 m) between devices or turn off the mobile phone.
- (18). The device is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- (19). Medical electrical equipment needs special precautions regarding EMC and needs to be installed according to EMC information. Contact the home care provider regarding

EMC installation information.

- (20). Always comply with the local regulations for disposing of such equipment.
- (21). Store the product in a dry place. Keep out of direct sunlight. Use or store the product only in the following conditions:
  - Operation: 10°C to 40°C (50°F to 104°F); 10% to 90% RH, non-condensing
  - Storage: -15°C to 50°C (5°F to 122°F); 10% to 90% RH, non-condensing

### CAUTION – Reducing the risk of damage to the device.

- (1). Mobile RF communications equipment can affect medical electrical equipment.
- (2). Do not place the device directly onto flammable materials.
- (3). The equipment must not be covered or positioned in such a way that the operation or performance of the equipment is adversely affected.
- (4). Be careful not to place the device where it can be taken or touched by children and pets.

### NOTE, CAUTION, AND WARNING STATEMENTS:

**NOTE:** Indicates useful tips.

**⚠ CAUTION:** Indicates correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property.

**WARNING:** Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

### SYMBOLS

|                                                                                     |                                                                                         |                                                                                     |                                        |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|----------------------------------------|
|    | Authorized representative in the European community                                     |    | Refer to instruction manual or booklet |
|   | Professional dry cleaning in tetrachloroethane and all solvents listed for the symbol F |   | Keep dry                               |
|  | Type BF applied part                                                                    |  | Caution                                |
|  | Do Not Iron                                                                             |  | Class II equipment                     |
|  | Manufacturer                                                                            |  | Temperature limit                      |
|  | Do Not Bleach                                                                           |  | Humidity limitation                    |
|  | Tumble dry possible at low temperature; exhaust temperature max. 60°C (140°F)           |  | Do Not Dry Clean                       |

|                                                                                   |                                                                                                                                                                                                                                                                                                                                                                                                                                   |                                                                                   |                                            |
|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|--------------------------------------------|
|  | Medical Device                                                                                                                                                                                                                                                                                                                                                                                                                    |  | Unique device identifier                   |
|  | Fragile, handle with care                                                                                                                                                                                                                                                                                                                                                                                                         |  | Date of manufacture                        |
|  | Use no hand hooks                                                                                                                                                                                                                                                                                                                                                                                                                 |  | Use no hooks shipping symbol               |
|  | Machine wash, regular/normal, 95°C (203°F)                                                                                                                                                                                                                                                                                                                                                                                        |  | Do not use cutter                          |
|  | CE Marking                                                                                                                                                                                                                                                                                                                                                                                                                        |  | Machine wash, regular/normal, 60°C (140°F) |
|  | Power “ON”                                                                                                                                                                                                                                                                                                                                                                                                                        |  | Power “OFF”                                |
|  | This way up                                                                                                                                                                                                                                                                                                                                                                                                                       |  | Maximum/minimum user weight                |
| <b>IP21</b>                                                                       | Protected against solid foreign objects of 12,5 mm and greater; Protection against vertically falling water drops                                                                                                                                                                                                                                                                                                                 |  | Importer                                   |
|  | <p>Attention</p> <p>Observe proper Disposal of Electrical &amp; Electronic Equipment (WEEE): This product should be handed over to an appropriate collection point for the recycling of electrical and electronic equipment. For more detailed information about the recycling of this product, please contact your local city office, household waste disposal service or the retail store where you purchased this product.</p> |                                                                                   |                                            |

# 1. Introduction

This manual should be used for the initial setup of the system and reference purposes.

## 1.1. General Information

The system is a high-quality and affordable mattress system suitable for the treatment and prevention of pressure ulcers.

The system has been tested and successfully approved to the following standards:

IEC/EN 60601-1  
IEC/EN 60601-1-2  
IEC/EN 61000-3-2 Class A  
IEC/EN 61000-3-3  
CISPR 11 Group 1, Class B

## EMC Warning Statement

This equipment has been tested and found to comply with the limits for medical devices to EN 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment to an outlet on a circuit different from that to which the other device(s) are connected.
- Consult the manufacturer or field service technician for help.

## 1.2. Intended Use

The system is intended to prevent the development of pressure injuries in patients at risk. It can only be operated by personnel with adequate pressure injury prevention training in acute care settings, long-term care facilities, and home environments.

## 1.3. Indications for Use

The system is indicated for patients whose weights are within the system's specified minimum and maximum support weight limits.

## 1.4. Contraindications

Do not use on patients with unstable fractures, including unstable spinal fractures, or those undergoing cervical traction or skeletal traction.

## 2. Product Description


### 2.1 Content of Package

Unpack to check for any damage that may have occurred during shipment. If there is damage, get in touch with your service provider immediately.

This system comes with the following items.

- (1). Pump Unit x1
- (2). Mattress x1
- (3). Power cord x1 (16 ft)
- (4). User Manual x1


### 2.2. Pump Unit

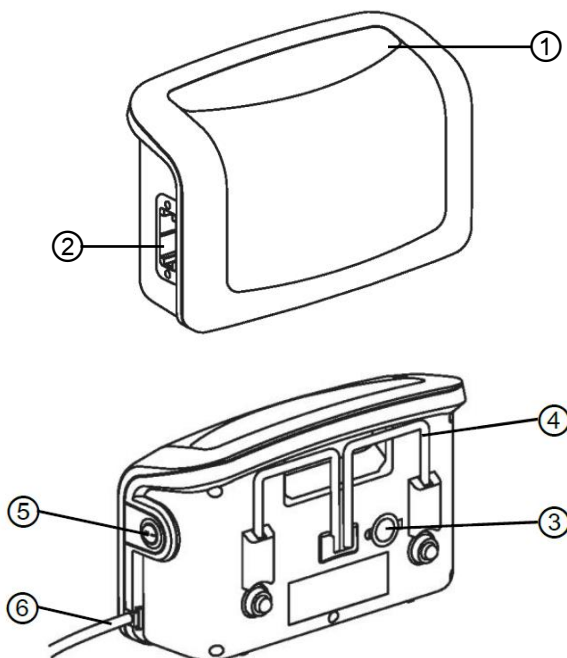- |                           |                        |
|---------------------------|------------------------|
| (1). Panel                | (4). Mounting Brackets |
| (2). Quick Connector Port | (5). Power Switch      |
| (3). Filter Cover         | (6). Power Cord        |

## 2.3. Panel
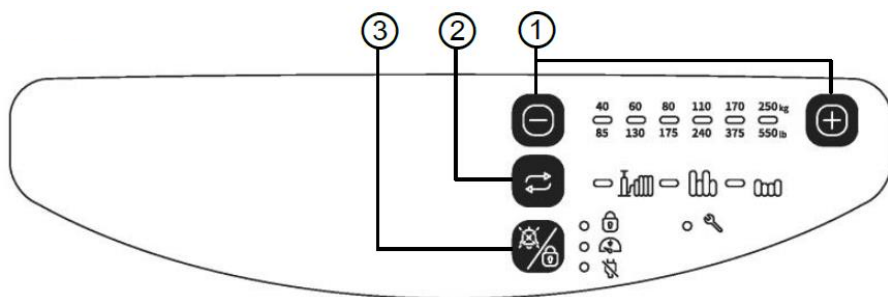
### (1) Pressure Adjustment Buttons

Press to adjust the mattress pressure according to the patient's weight.

### (2) Mode Button

Press to select the preferred treatment mode. The default is alternating mode.

---

#### Max Firm

To inflate the mattress to ensure that maximal operating pressure is available. This mode lasts for 20 minutes and then automatically returns to the previous mode. This mode provides solid and firm support for the patient and is optimal for patient transfers or care practices.

- Press to inflate the mattress.
- Press again to stop inflation. The system will then return to the previous mode.

---

#### Alternating (ALT)

The odd and even rows of air cells take turns to inflate and deflate, creating a wave-shaped surface that aims to relieve surface pressure periodically.

---

#### Continuous Low Pressure (CLP)

Air cells don't alternate and are equally inflated at a lower air pressure, creating a flat surface that aims to distribute the patient's weight by maximizing contact area.

---

### (3) Mute/Panel Lock Button

Audible and visual alarms (solid orange indicator) are provided to alert users. Press here to silence the audible alarm, and the indicator will change to flash. The alarm will resume after 3 minutes if the problem persists.

---

#### Panel Lock

Press to lock/unlock the control panel.

The panel locks automatically 5 minutes after the last operation. This prevents the settings from being accidentally changed.

---

#### Low Pressure Alarm

The pressure inside the mattress is lower than normal. Refer to troubleshooting for details.

---

**Power Failure Alarm (PFA)**

Power source not detected.

**NOTE:** The pump seals the air inside the mattress, which temporarily keeps the patient from bottoming out during a power failure. Move the patient elsewhere if the situation cannot be resolved quickly.

**Service Alarm**


Possible system failure.

Contact the service provider for assistance.

● **Indicator Status**

Indicators are shown in two colors: green and orange. See below for status.
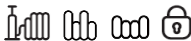
| Icon     |                                                      |  |  |
|----------|---------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|
| Color    | Green                                                                                                                                 | Orange                                                                            |                                                                                   |
| Status   |                                                                                                                                       |                                                                                   |                                                                                   |
| Solid    | <ul style="list-style-type: none"> <li>● Features enabled</li> <li>● Inflation in progress</li> <li>● Control panel locked</li> </ul> | N/A                                                                               | Errors. See section 9 for details.                                                |
| Flashing | N/A                                                                                                                                   | Errors. See section 9 for details.                                                | N/A                                                                               |
| Dim      | Disabled                                                                                                                              | Normal operation                                                                  |                                                                                   |


**2.3.1. Modes**

An active support surface is a powered support surface that has the ability to change its load distribution properties with or without an applied load.

A reactive support surface is a powered or non-powered support surface with the ability to change its load distribution properties only in response to an applied load. (Refer to international guideline 2019: prevention and treatment of pressure ulcers/injuries: clinical practice guideline)


|             | Active  | Reactive |
|-------------|---------|----------|
| Non-powered | X       | ✓        |
| Powered     | ✓ (ALT) | ✓ (CLP)  |

## 2.4. Mattress

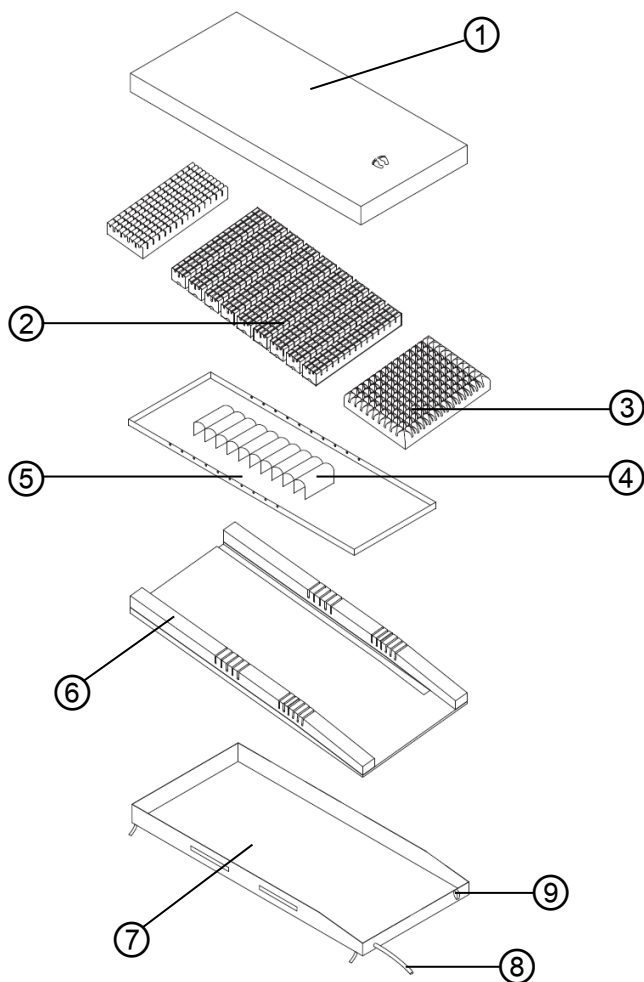- |                               |                           |
|-------------------------------|---------------------------|
| (1). Top cover                | (6). Supporting edge foam |
| (2). Air cell with inner foam | (7). Bottom cover         |
| (3). Heel foam                | (8). Air hose             |
| (4). Air cell sleeve          | (9). Air hose loop        |
| (5). Air cell base            |                           |

### 3. Installation
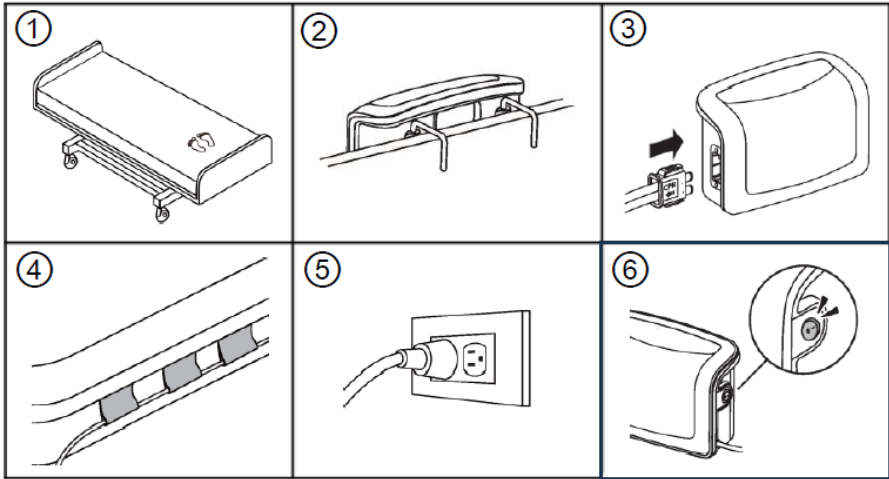
#### 3.1. Installation of pump and mattress

- (1) Place the mattress or pad on top of a bed frame. Orient the mattress so that the foot-markers on the mattress are situated towards the end of the bed (where the pump will be located).

**NOTE:** The nurse should assess the patient's condition to determine whether the pump is required. The Crossover Nio mattress can be used on its own.

**NOTE:** When the mattress is used on its own, it is recommended to keep the quick connector's cap closed while the patient is lying on the bed. After approximately 14 days, the patient may get out of bed, and the cap can be opened to allow air to refill the mattress.

- (2) Hang the pump from the bed panel at the foot end.
- (3) Remove the quick connector's cover. Connect the mattress's air hose to the pump. To ensure the connection is made, you should hear/feel a "click".

**WARNING:** To prevent tripping, place the air hose away from walking areas.

**NOTE:** Make sure that the air hose is not kinked or tucked under the mattress.

- (4) The cable management tape is provided along the side of the mattress to manage the power cord. After installation, any extra length of power cord should be neatly tucked away to prevent tripping and away from moving bed mechanisms or other potential entrapment points.

**NOTE:** Do not position the device so that it is difficult to disconnect the plug.

**WARNING:** Please make sure that the power cord does not show any signs

**of damage.**

- (5) Plug the power cord into the mains power.

**NOTE: The pump unit must be suitable for the local power voltage.**

**NOTE: The plug can also be used to disconnect the device.**

- (6) Turn the main power switch (on the pump) to the ON ("I") position. After turning on the pump, it will automatically go into Therapy mode (ALT or CLP mode).

**NOTE: You can unplug the unit to disconnect the device. Do not position the equipment so that it is difficult to operate the disconnecting device.**

**⚠ CAUTION: The pump supplied with the mattress can only be used for mattresses recommended by the manufacturer. Do not use it for any other purpose. (applied part: air mattress)**

**NOTE: During power outages, you can cover the Quick Connector with the transport cap to maintain air pressure inside the air cells.**

**NOTE: After installation, make sure any excess cord or tubing is removed from possible foot traffic to avoid accidental tripping. All EQUIPMENT should be positioned to always allow unhindered patient access by physicians and caregivers.**

## 4. Operation

**NOTE: Always read the instructions before use.**

### 4.1. General Operation

- (1). Switch on the power switch.
- (2). Press the Power switch to activate the system. The system will be fully inflated.
- (3). During inflation, press the Mode button to select a preferred mode.

### 4.2. Pressure Adjustment

Use the pressure adjustment buttons to control the air pressure in the mattress. Press “+” to make the mattress firmer. Press “-” to make the mattress softer. Higher pressures will support heavier patients. The pressure should be adjusted according to the individual's weight.

**NOTE: You can also check if the pressure is suitable for the patient by sliding one hand beneath the air cells at the level of the patient's buttocks. The air cells will alternately inflate and deflate. You should feel slight contact with the buttocks when the air cells beneath the buttocks deflate.**

### 4.3. Mode Operation

The Mode button switches among the Max Firm, Alternating, and Continuous Low Pressure modes. In the Max Firm mode, the air cells inflate to maximum working pressure to provide solid and firm support, ideal for patient transfers or care practices. In alternating mode, the air cells are partially deflated and inflated, avoiding prolonged pressure on any single point beneath the patient. This is to prevent pressure ulcers. In continuous low-pressure mode, all of the air cells are equally inflated to provide a more comfortable and even surface for the patient.

### 4.4. Indicators

Please refer to Section 2.3(3) for the Indicator Status table.


**NOTE: If the pressure level is consistently low, check for any leakage (tubes or air hoses). If necessary, replace any damaged tubes or hoses, or contact your local qualified dealer for repair.**

### 4.5. Alarm Mute Operation

The pump unit is equipped with an audible alarm that activates when an alert condition occurs, accompanied by the corresponding indicator light. When the alarm sounds, the caregiver may press the Mute button to temporarily silence the alarm while addressing the issue. Please refer to the Troubleshooting section for further guidance.

**NOTE: Press the mute button again to re-activate the audible alarm function.**

#### 4.6. CPR

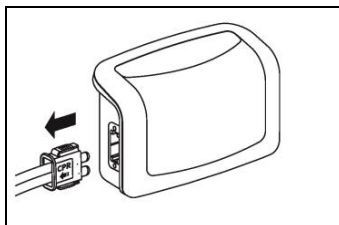When there is an emergency to perform CPR on the patient, quickly unplug the Quick Connector to release air from the mattress.

## 5. Cleaning

It is important to follow the cleaning procedures to avoid cross-infection. Ensure to clean in a dry and dust-free environment.

**WARNING: To prevent cross-infection, always clean and disinfect the system after each patient.**

### 5.1 Before Cleaning

- (1). Turn off the system.
- (2). Unplug the device.
- (3). Pull off the air hose from the pump unit.
- (4). Dilute the detergent according to its manufacturer's instructions to prepare the cleaning solution. Using a mild detergent with a pH between 6 and 8 is recommended.

**⚠CAUTION: Never apply detergents directly to the pump or mattress as this might affect performance.**

**WARNING: Do not use detergents that may deposit calcium carbonate.**

**WARNING: Do not use detergents or disinfectants that may damage the plastic casting, such as benzene, toluene, phenol, acetone, 70% isopropyl alcohol (IPA), or others.**

**⚠CAUTION: Do not use phenolic-based products for cleaning.**

### 5.2 General Cleaning

General cleaning should be carried out at least once a week to maintain hygiene.

#### ● Pump Unit

- (1). Dampen a clean cloth with the cleaning solution. Wipe clean the pump unit, particularly its panel, label or hanger, to remove any visible dirt.
- (2). Wipe again with water about 18 to 40°C (64.4 to 104°F) to remove any remaining detergent. Repeat this step if necessary to ensure a detergent-free finish.
- (3). Dry with a dry cloth.
- (4). Allow to air dry. Keep out of direct sunlight.

**⚠CAUTION: Do not immerse or soak the pump unit.**

**⚠CAUTION: Make sure that no liquid gets inside the pump.**

**⚠CAUTION: The pump does not need oil lubrication.**

**WARNING: Do not remove the pump housing to avoid electrical shock.**

**WARNING: Never disassemble the system by yourself. All disassembly or repair should be done by professional technicians.**

- **Mattress**

- (1). Unzip to separate the top cover, cell parts, and bottom cover.
- (2). Check for visible damage or possible leaks. Replace if found leaking or damaged.
- (3). Dampen a clean cloth with the cleaning solution and wipe the surface clean.
  - Particularly the top cover, the bottom cover, or the straps, to remove any visible dirt.
  - Unzip the top cover and bottom cover to wipe clean every cell inside.
- (4). Wipe again with fresh water about 18 to 40°C (64.4 to 104°F) to remove any remaining detergent. Repeat this step if necessary to ensure a detergent-free finish.
- (5). Dry with a dry cloth.
- (6). Allow to air dry. Keep out of direct sunlight.

**⚠ CAUTION: Keep out of direct sunlight. Dry before use.**

**WARNING: Wipe gently! Do not scrub.**

### 5.3 Advanced Cleaning

Advance cleaning should be carried out between patients to avoid cross infection as the product is in direct contact with the patient.

|                        |                                                                                    |
|------------------------|------------------------------------------------------------------------------------|
| Top Cover: Stretch     |   |
| Bottom Cover: Nylon/PU |  |

- **Top Cover/bottom cover**

- (1) Put to machine laundry. Do not exceed the limit specified below.
  - ✓ With detergent:  
At temperatures below 60°C (140°F) for 10 to 15 minutes.
  - ✓ Without detergent:
    - Top Cover: At temperatures below 95°C (203°F) for 10 to 15 minutes.
    - Bottom Cover: At temperatures below 60°C (140°F) for 10 to 15 minutes.
- (2) Tumble dry at a medium-low temperature of 40°C/104°F for 50 minutes.

**WARNING: Please follow the cleaning instructions. Do not exceed the limit.**

- **Cell parts**

Cell parts include the air cell and the air cell base.

- (1) Dampen a clean cloth with the cleaning solution and wipe clean the cell, to remove any visible dirt.
- (2) Wipe again with fresh water between 18 to 40°C (64.4 to 104°F) to remove any remaining detergent. Repeat this step if necessary to ensure a detergent-free finish.
- (3) Dry with a dry cloth.
- (4) Allow to air dry. Keep out of direct sunlight.

**WARNING: Never machine wash the cell parts.**

## **5.4 Disinfection in Healthcare Settings**

Disinfection should be carried out after each patient to avoid cross infection. Disinfect only after thorough cleaning.

**WARNING: Do not disinfect in the presence of patients to avoid eye or skin irritations.**

**⚠ CAUTION: Do not autoclave or boil any part of the product.**

**⚠ CAUTION: Always wipe clean again with fresh water. This ensures that no residue remains.**

**WARNING: Dry before use.**

- **Before disinfection**

- (1). Conduct advance cleaning. Please refer to section 5.3 for more information.
- (2). Prepare a damp cloth pre-soaked with disinfectants. Recommended disinfectants are listed below.

- 75% alcohol solution

- Sodium hypochlorite solution with 5,000 to 10,000 ppm

**⚠ CAUTION: Never apply disinfectant directly to the pump or mattress.**

**⚠ CAUTION: Do not use sodium hypochlorite solution with a concentration higher than 10,000ppm as prolonged use may reduce the performance of the coating and shorten the life of the product.**

**⚠ CAUTION: Do not use 70% Isopropyl alcohol (IPA) for disinfection.**

**⚠ CAUTION: Do not use ozone or UV for disinfection.**

- **Pump unit**

- (1). Wipe clean the pump unit, particularly the casting, panel, label, and

hanger.

- (2). Wait for the disinfectant to take effect according to the instructions.
- (3). Wipe again with water about 18 to 40°C (64.4 to 104°F) to remove the remaining disinfectants. Repeat this step if necessary to ensure a disinfectant-free finish.
- (4). Dry with a dry cloth.
- (5). Allow to air dry. Keep out of direct sunlight.

**WARNING: To avoid electrical shock, always unplug the power cord from the mains power before disinfection.**

**⚠ CAUTION: Make sure that no liquid gets inside the pump.**

- **Mattress**

- (1). Wipe clean, particularly the straps.
- (2). Unzip the top cover and bottom sheet to wipe the cell parts inside for thorough disinfection.
- (3). Wait for the disinfectant to take effect according to the instructions.
- (4). Wipe again with water about 18 to 40°C (64.4 to 104°F) to remove the remaining disinfectants. Repeat this step if necessary to ensure a disinfectant-free finish.
- (5). Dry with a dry cloth.
- (6). Allow to air dry. Keep out of direct sunlight. Dry before use.

**WARNING: Wipe gently! Do not scrub.**

## 6. Storage


- (1). Lay the mattress flat.
- (2). Place the cap on the Quick Connector and insert it through the air hose loop. Store the mattress in this flat position.

**⚠ CAUTION: Do not kink, crease, or stack the mattresses, and do not store the system in direct sunlight, high temperatures, or moist areas.**

## 7. Maintenance

- (1) Check the main power cord and plug for abrasions or excessive wear.
- (2) Check the cover for signs of wear or damage. Ensure the mattress cover and tubes are stubbed together correctly.
- (3) Check the airflow from the quick connector. The airflow should alternate between each connector every half-cycle time if it's in alternating mode.
- (4) Check the air hoses for kinks or breaks. For replacement, please contact our local dealers.

### 7.1. Air Filter Replacement

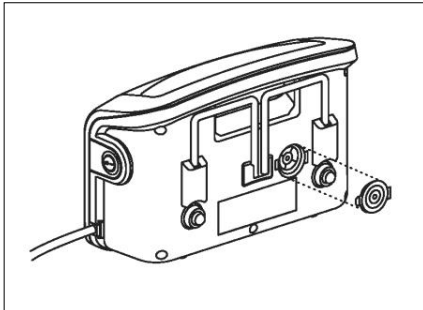- (1) Open the air filter cap located at the back of the pump.
- (2) The filter is reusable and can be washed gently with a mild detergent and water. Dry the filter before use.
- (3) Check and replace the air filter regularly if the environment is dirty.

## 8. Disposal

Always comply with the local regulations for disposing of such equipment.

## 9. Troubleshooting

| ICON                                                                              | MEANING                                                            | WHAT TO DO                                                                                                                                                                                                                                                                             |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Possible power failures                                            | <ul style="list-style-type: none"> <li>● Check if the plug is connected to the mains power.</li> <li>● Check the switch on/off position.</li> <li>● Check if there is a power outage.</li> <li>● Check if the power cord is connected to the pump properly.</li> </ul>                 |
|  | Possible system failure.                                           | <ul style="list-style-type: none"> <li>● Stop using the system and move the patient onto another support surface.</li> <li>● Call customer service for further assistance.</li> </ul>                                                                                                  |
|  | The pressure does not reach the desired values for a certain time. | <ul style="list-style-type: none"> <li>● Check if the connection between the quick connector and to pump unit is tightly secured.</li> <li>● Check if all tubing connections along the mattress are secured.</li> <li>● Check if there is any air leakage in the air cells.</li> </ul> |

If the above information does not solve your problems, please contact your local agent directly. They might require a technician to take care of the problem.

## 10. Technical Specifications

| Item                                                        |                             | Specification                                                                                                                                                          |
|-------------------------------------------------------------|-----------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Power Supply (Note: See rating label on the product)</b> |                             | AC 220-240V, 50 Hz, 0.08 A                                                                                                                                             |
| <b>Fuse Rating</b>                                          |                             | T1AL, 250V                                                                                                                                                             |
| <b>Cycle Time</b>                                           |                             | 10 min @ 50Hz                                                                                                                                                          |
| <b>Dimension (L x W x H)</b>                                |                             | 29 x 18.5 x 12.6 cm / 11.4" x 7.3" x 5"                                                                                                                                |
| <b>Weight</b>                                               |                             | 2.2 kg / 5 lb                                                                                                                                                          |
| <b>Environment</b>                                          | <b>Atmospheric Pressure</b> | 700 hPa to 1060 hPa                                                                                                                                                    |
|                                                             | <b>Temperature</b>          | Operation: 10°C to 40°C (50°F to 104°F)<br>Storage: -15°C to 50°C (5°F to 122°F)<br>Shipping: -15°C to 70°C (5°F to 158°F)                                             |
|                                                             | <b>Humidity</b>             | Operation: 10% to 90% non-condensing<br>Storage: 10% to 90% non-condensing<br>Shipping: 10% to 90% non-condensing                                                      |
| <b>Classification</b>                                       |                             | Class II, Type BF, IP21<br>Applied Part: Air Mattress<br><b>NOTE: Not suitable for use in the presence of a flammable anesthetic mixture (No AP or APG protection)</b> |
| <b>Service Life</b>                                         |                             | System: 7 years                                                                                                                                                        |
| <b>Operating Noise</b>                                      |                             | < 30 dBA                                                                                                                                                               |
| Mattress                                                    |                             | Specification                                                                                                                                                          |
| <b>Model</b>                                                |                             | 6" Replacement                                                                                                                                                         |
| <b>Flame Retardant Standards</b>                            |                             | BS7177, Medium Hazard                                                                                                                                                  |
| <b>Dimensions (L x W x H)</b>                               |                             | 200 x 90 x 15 cm / 78.7" x 35.4" x 6"<br>190 x 80 x 15 cm / 74.8" x 31.5" x 6"                                                                                         |
| <b>Max. Support Weight</b>                                  |                             | 250 kg / 550 lb                                                                                                                                                        |
| <b>Min. Support Weight</b>                                  |                             | 40 kg / 88 lb                                                                                                                                                          |
| <b>Weight</b>                                               |                             | 15 kg / 33.75 lb                                                                                                                                                       |

### NOTE:

1. Consult the distributor or EU representative for further technical documents.
2. These specifications are also applicable to other regions operating with the same power supply.
3. The manufacturer reserves the right to modify the specifications without notice.

## 11. Servicing

The products are intended to offer safe and reliable operation when used or installed according to the instructions provided by Wellell. Wellell recommends that the system be inspected and serviced by authorized technicians if there are any signs of wear or concerns with device function and indication on products. Otherwise, service and inspection of the devices generally should not be required.

## 12. Conditions of Warranty

- (1) The user must follow all instructions provided by Wellell Inc. for proper cleaning and maintenance of the mattress.
- (2) The internal and external components of the product must be inspected at least once every three (3) months for signs of damage or wear. Any damaged components must be promptly replaced with Wellell-approved replacement parts. If any part is found to be defective or damaged, the mattress must be immediately removed from service until the appropriate replacement has been completed.

This warranty is limited and excludes all other warranties, express or implied. Without limiting the generality of the foregoing, Wellell Inc. shall not be liable for the following:

- (1) Damage resulting from conditions beyond the control of Wellell, including but not limited to: negligence (whether intentional or unintentional), accidental damage, impact by sharp objects, misuse, tampering, fire, or damage incurred during transport after leaving Wellell's facilities. This also includes damage from excessive wear and tear, failure to follow cleaning instructions, use of inappropriate or improperly diluted cleaning agents or processes, and failure to ensure proper drying of the mattress before turning or flipping.
- (2) Damage arising from contamination after product damage or failure to replace the protective cover. This includes damage caused by improper storage, failure to inspect or maintain the mattress or cushion interior, or exposure to unusual contamination, infections, spills, stains, or corrosive or abrasive substances.
- (3) Damage caused by using the product outside its intended purpose or contrary to its recommended use.
- (4) Damage due to failure to replace defective parts in a timely manner, or the use of parts not supplied or approved by Wellell or its authorized agents. This includes damage from operating the product without a cover, with a damaged cover, or with a cover that is improperly fitted.
- (5) Damage resulting from integrating the mattress with other products or equipment not specifically approved by Wellell.
- (6) Any indirect, incidental, or consequential damages arising from the use of the mattress or cushion.
- (7) Any liability exceeding the original purchase price of the mattress or cushion.
- (8) Claims based on dissatisfaction with the product that are not related to defects in materials, workmanship, or assembly.

# Appendix A: EMC Information

## Guidance and Manufacturer's Declaration- Electromagnetic Emissions:

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

| Emissions Test                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         | Compliance | Electromagnetic Environment-Guidance                                                                                                                                                                                                                                                                                                                      |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| RF emissions<br>CISPR 11                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               | Group1     | The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment<br><br>The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network |
| RF emissions<br>CISPR 11                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               | Class B    |                                                                                                                                                                                                                                                                                                                                                           |
| Harmonic emissions<br>IEC61000-3-2                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     | Class A    |                                                                                                                                                                                                                                                                                                                                                           |
| Voltage fluctuations / Flicker emissions<br>IEC61000-3-3                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               | Complies   |                                                                                                                                                                                                                                                                                                                                                           |
| <div><div>Warning:</div><div><div>(1)</div><div>The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.</div></div><div><div>(2)</div><div>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</div></div><div><div>(3)</div><div>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Pump, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</div></div></div> |            |                                                                                                                                                                                                                                                                                                                                                           |

## Guidance and Manufacturer's Declaration- Electromagnetic Immunity:

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

| Immunity Test Levels  |                                            |                                       |                      |                                         |
|-----------------------|--------------------------------------------|---------------------------------------|----------------------|-----------------------------------------|
| Basic EMC<br>standard | Professional<br>healthcare<br><br>facility | HOME<br>HEALTHCARE<br><br>ENVIRONMENT | Compliance<br>Levels | Electromagnetic<br>Environment-Guidance |

| environment                                                                                           |                                                                                                                                                                                               |                                                                                                                  |                                                                                                                  |                                                                                                                                                                                                                                                                               |
|-------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Electrostatic Discharge (ESD)<br>IEC61000-4-2                                                         | ±8kV contact<br>±2, 4, 8, 15 kV air                                                                                                                                                           |                                                                                                                  | ±8kV contact<br>±2, 4, 8, 15 kV air                                                                              | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.                                                                                                                                |
| Electrical fast transient/ burst<br>IEC61000-4-4                                                      | ±2 kV, 100 kHz for power supply line<br>±1 kV, 100 kHz for input/output line                                                                                                                  |                                                                                                                  | ±2 kV, 100 kHz for power supply line                                                                             | Mains power quality should be that of atypical commercial or hospital environment                                                                                                                                                                                             |
| Surge<br>IEC61000-4-5                                                                                 | ±0.5, 1 kV (line to line)<br>±0.5, 1, 2 kV (line to earth)                                                                                                                                    |                                                                                                                  | ±0.5, 1 kV (line to line)                                                                                        | Mains power quality should be that of atypical commercial or hospital environment.                                                                                                                                                                                            |
| Voltage dips, short interruptions and voltage variations on power supply input lines<br>IEC61000-4-11 | Voltage Dips:<br>i) 100% reduction for 0.5 period,<br>ii) 100% reduction for 1 period,<br>iii) 30% reduction for 25/30 period,<br>Voltage Interruptions:<br>100% reduction for 250/300 period |                                                                                                                  | 230V                                                                                                             | Mains power quality should be that of a typical commercial or hospital environment. If the user of this device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery. |
| Power frequency (50/60Hz) magnetic field<br>IEC61000-4-8                                              | 30 A/m                                                                                                                                                                                        | 30 A/m                                                                                                           | 30 A/m                                                                                                           | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.                                                                                                                                     |
| Conducted RF<br>IEC 61000-4-6                                                                         | 3 Vrms<br>0,15 MHz – 80 MHz<br>6 Vrms in ISM bands between 0,15 MHz and 80 MHz<br>80 % AM at 1 kHz                                                                                            | 3 Vrms<br>0,15 MHz – 80 MHz<br>6 Vrms in ISM and amateur radio bands between 0,15 MHz and 80 MHz<br>80 % AM at 1 | 3 Vrms<br>0,15 MHz – 80 MHz<br>6 Vrms in ISM and amateur radio bands between 0,15 MHz and 80 MHz<br>80 % AM at 1 | Portable and mobile RF communications equipment should be used no closer to any part of this device, including cables, than there commended separation distance calculated from the equation applicable to the frequency of the transmitter.                                  |

|                                    |                                                                                                                                          |                                                                                                                                           |                                                                                                                                             |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
|------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                    |                                                                                                                                          | kHz                                                                                                                                       | kHz                                                                                                                                         | Recommended separation distance                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
|                                    |                                                                                                                                          |                                                                                                                                           |                                                                                                                                             | $d = \sqrt{P}$ 150kHz to 80MHz<br>$d = 0.6 \sqrt{P}$ 80MHz to 800MHz<br>$d = 1.2 \sqrt{P}$ 800 MHz to 2.7GHz                                                                                                                                                                                                                                                                                                                                                                                                                                                  |
| Radiated EM Fields<br>IEC61000-4-3 | 3 V/m 80 MHz to 2.7 GHz<br>80 % AM at 1 kHz<br>380 - 5800 MHz,<br>9-28 V/m,<br>Pulse modulation or FM $\pm 5$ kHz deviation, (1kHz sine) | 10 V/m 80 MHz to 2,7 GHz<br>80 % AM at 1 kHz<br>380 - 5800 MHz,<br>9-28 V/m,<br>Pulse modulation or FM $\pm 5$ kHz deviation, (1kHz sine) | 10 V/m<br>80 MHz - 2.7 GHz<br>80 % AM at 1 kHz<br>380 - 5800 MHz,<br>9-28 V/m,<br>Pulse modulation or FM $\pm 5$ kHz deviation, (1kHz sine) | Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). <sup>b</sup><br>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency ranged.<br>Interference may occur in the vicinity of equipment marked with the following symbol:<br> |

NOTE 1: UT is the a.c. mains voltage prior to the application of the test level

NOTE 2: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land

mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level

above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

### Recommended separation distances between portable and mobile RF communications equipment and this device:

This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment

| Rated maximum<br>output power<br>of transmitter<br><br>W | Separation distance according to frequency of transmitter m |                   |                    |
|----------------------------------------------------------|-------------------------------------------------------------|-------------------|--------------------|
|                                                          | 150 kHz to 80 MHz                                           | 80 MHz to 800 MHz | 800 MHz to 2,7 GHz |
|                                                          | $d = \sqrt{P}$                                              | $d = 0.6\sqrt{P}$ | $d = 1.2\sqrt{P}$  |
| 0.01                                                     | 0.1                                                         | 0.06              | 0.12               |
| 0.1                                                      | 0.31                                                        | 0.19              | 0.38               |
| 1                                                        | 1                                                           | 0.6               | 1.2                |
| 10                                                       | 3.1                                                         | 1.9               | 3.8                |
| 100                                                      | 10                                                          | 6                 | 12                 |

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

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All rights reserved
**Apex Medical (Kunshan) Corp.**